

# SPAN AMERICA PUMPS

## Custom Care Convertible Pump LAL



### THERAPY MODE:

1. Float mode will evenly inflate all cylinders.
2. Alternating mode will alternate inflating cylinders 1 and 3 to inflating cylinders 2 and 4 every 15 minutes.
3. Rotation mode will inflate cylinders 1 and 2 alternating with cylinders 3 and 4 to provide approximately 20 degrees of lateral rotation every 15 minutes. In order to achieve rotation with the APM2 there is a switch where the hose meets the mattress. ***Lateral rotation should not be used when the head of the bed is above 30 degrees.***

Maximum inflation should be used for transfers, ADLs and CPR, achieve this by using float mode with comfort level 5.

**AIR FLOW:** Provides low air loss feature for micro climate control.

**COMFORT LEVEL:** Can be adjusted for clients comfort. Clients over 120 lbs should not use level one.

**LOW PRESSURE INDICATOR:** Will light up upon initial set up and may stay on for up to 30 minutes. If light doesn't go out after 30 minutes-service may be required

**Mattress is READY for USE when** NORMAL PRESSURE indicator lights up and low pressure indicator goes off

**LOCK OUT :** holding this button for 3 to 4 seconds will lock the options on the pump. Hold the LOCK OUT button for another 3 to 4 seconds to unlock options.

**Power Failure:** if power is lost, do not unplug the control unit or disconnect air lines from the control unit.. The air cylinders will equalize pressure in the cylinders and maintain inflation .